



## PROXIMA® 1 PROXIMA® 2


PROVIDING & CONNECTING PEOPLE WITH HIGH QUALITY STANDARDS

# Proxima® | Proxima® +

## Enjoying comfort, fulfilling your needs



### Types of backing material

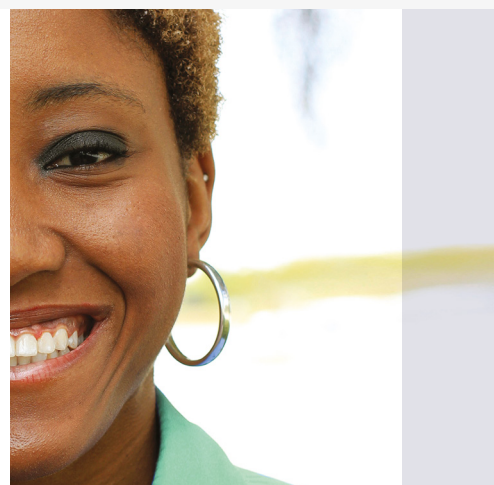

A complete cost-effective range to answer patients' needs

1-Piece		 <b>NEW</b>	2-Piece		 <b>NEW</b>	
					 <b>NEW</b>	

### How to apply a 2-Piece system?

1



Place the base plate around the stoma and fill, if needed, with paste where the protector does not cover the skin. To ensure perfect adhesion onto the skin, warm the skin protector using your fingers.

2



Connect the pouch to the base plate: hold the locking system up and place the base plate from the bottom, pressing with your finger along the ring to the top. You will hear a clicking noise when the pouch is in place.


3



When you hear the clicking noise, close the locking system by pushing at the bottom.

# Proxima<sup>®</sup> 1-Piece & 2-Piece Closed Pouches for colostomates

- Compact design for optimal comfort and discretion
- Integrated laminar filter - 40mm
  - To reduce odour and pouch ballooning
  - Completely integrated to decrease the risk of staining clothes
- Odour proof plastic film
  - For more discretion
- Available in beige for more discretion, and transparent for visual control after surgery
- Cut-to-fit up to 70mm

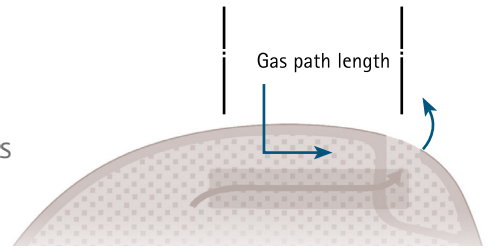

## 1-Piece pouches



### Proxima<sup>®</sup>


- Washable micro perforated material on the skin side of the pouch



### Proxima<sup>®</sup> + **NEW**


- Soft non-woven material on both sides of the pouch
- New with pre-cut sizes: 25 / 30 / 35 / 40 / 45 mm

## 2-Piece pouches

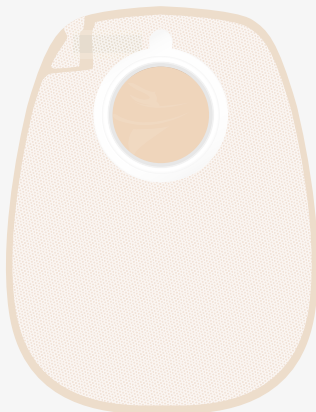

### Proxima<sup>®</sup> 2

- Washable micro perforated material on the skin side of the pouch

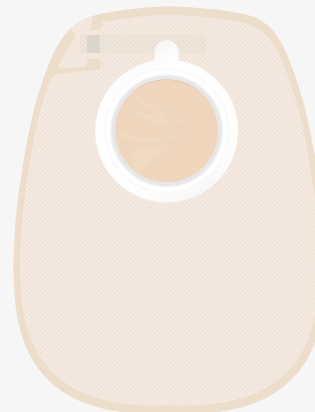


### Proxima<sup>®</sup> 2+ **NEW**

- Soft non-woven material on both sides of the pouch




# Proxima<sup>®</sup> 1-Piece & 2-Piece Drainable Pouches for ileostomates and colostomates


- Integrated laminar filter - 60mm
  - To reduce odour
  - Completely integrated to decrease the risk of staining clothes
  - Available with or without filter
- Large capacity:  
650ml for 1-Piece pouches & 720ml for 2-Piece pouches (ø 60)
- Large drainable outlet - 7cm
  - For ileostomy or colostomy stools evacuation
  - Easier cleaning of the outlet and emptying of the pouch
  - Reusable rigid clamp or single use soft adhesive clamp available
- Odour proof plastic film
  - For more discretion

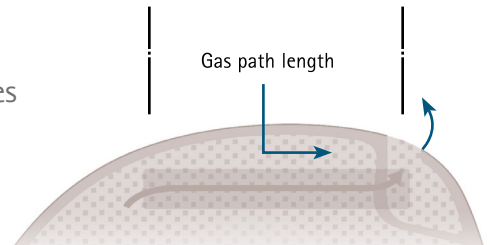


## How to use the Soft clamp?

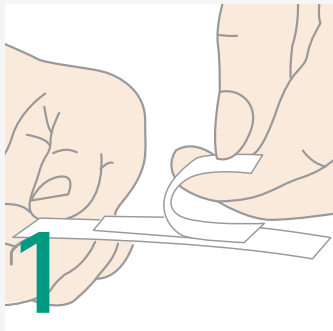


1 Remove the release paper from the adhesive strip.

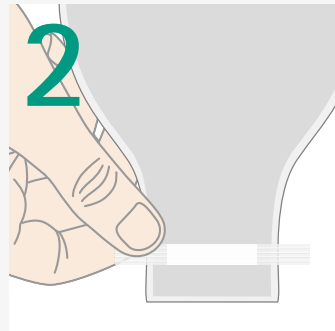

2 Centre and stick the soft clamp at the bottom of the opening (on the outer side of the pouch, not on the skin side). Leave a small space between the clamp and the end of the opening. Its size can be the width of the clamp.



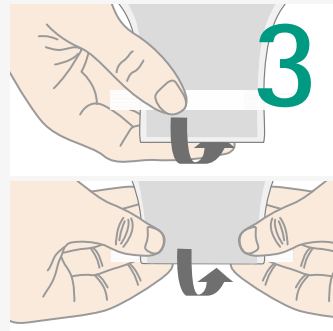


3 Fold the outlet backwards four times using the clamp.

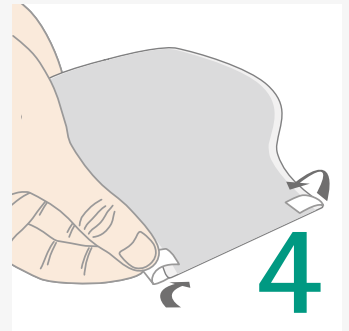

4 Squeeze the ends of the clamp firmly to complete closure.

## How to use the Rigid clamp?

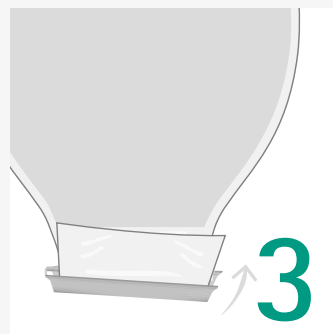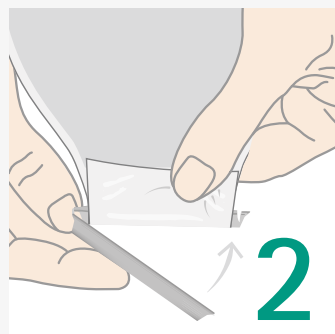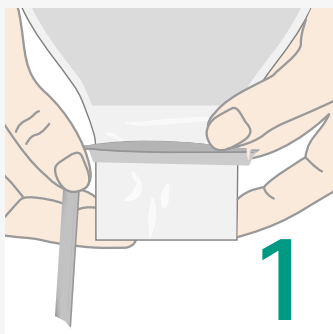


Only 3 steps:

1. Put the thin curved part of the clamp towards the plastic of the outlet
2. Fold the film upwards
3. Close the clamp


# Drainable Pouches for ileostomates and colostomates

## 1-Piece drainable pouches

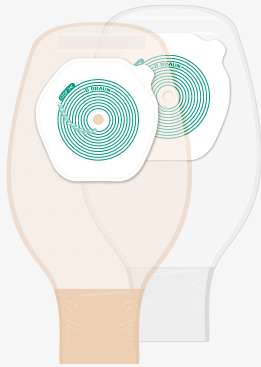


Proxima®

- 100% hydrocolloid skin protector
- Washable micro perforated material on the skin side of the pouch

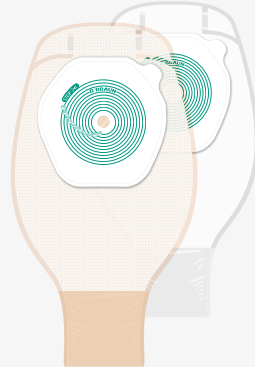

Proxima®+

- 100% hydrocolloid skin protector
- Soft non-woven material on the both sides of the pouch
- Available with or without filter

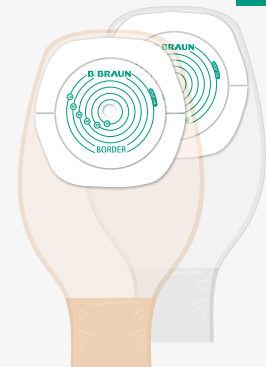


Proxima® Border

- Hydrocolloid skin protector with acrylic tape border
- Extended wear time: from 3 to 5 days\*
- Washable micro perforated material on the skin side of the pouch


\*Data on file ; Survey Chile.

## 2-Piece drainable pouches

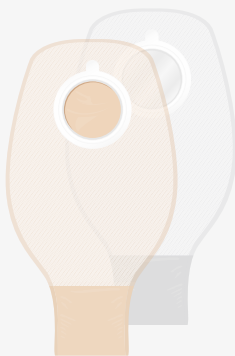


Proxima® 2

- Washable micro perforated material on the skin side of the pouch

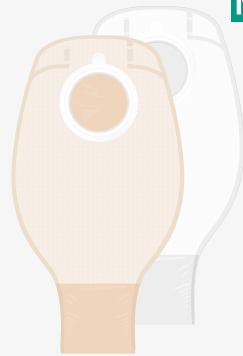

Proxima® 2+

- Soft non-woven material on the both sides of the pouch

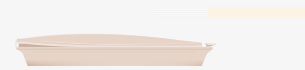


2 clamps

- Soft clamps for more comfort
- Rigid reusable clamp

# Proxima<sup>®</sup> 1-Piece & 2-Piece Uro Pouches for urostomates

- Compact design and optimized capacity for more discretion
- Anti-reflux valve
  - Prevents urine backflow which could lead to urinary infections
- Unique B. Braun outlet, secure & easy to handle
  - Hygienic; no residual drop on fingers

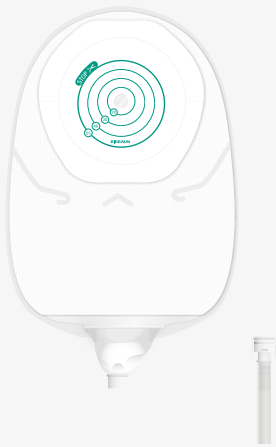

## Proxima<sup>®</sup> **NEW**

- Capacity: 565ml
- Universal connector very secure and easy to put on and to take off



## Proxima<sup>®</sup> 2 **NEW DESIGN**

- Capacity: 690ml (ø 60)
- Universal connector very secure and easy to put on and to take off

## A Secure Outlet

### OUTLET FOR DAY

- Clear position open/closed
- Fits in perfectly and is strengthened by an audible click and a final locking system
- Easy to handle; no risk of wet fingers




Closed



Open

### OUTLET FOR NIGHT

- Audible click at connection
- Easy to connect to a night drainage bag
- Outlet designed to avoid pouch twist

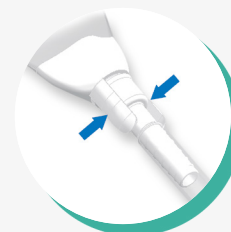

Universal Connection  
to urine bags



Capacity: 2L

# Proxima® I Proxima® +

## Common features & benefits

# 2

### Types of skin protectors

- Standard
- Extended wear time

### Shape of skin protector

- Designed to spread traction force out evenly

### Cuttable up to 70mm max


- Can be used in immediate post-op

## Skin protectors for 1-Piece system

### 1. Standard

#### 100% full hydrocolloid skin protector

- Optimal skin protection
- Flexibility and comfort
- Designed for frequent changes

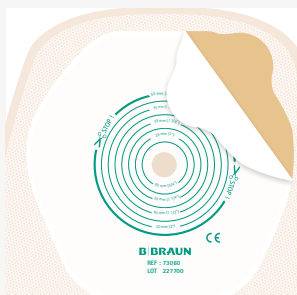


Standard skin protector

### 2. Border

#### Hydrocolloid with acrylic tape border skin protector

- Immediate adhesivity and secure feeling
- Increased wear time

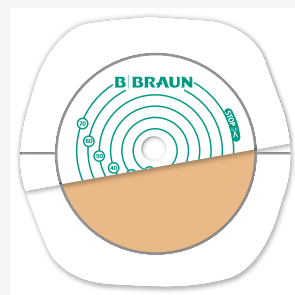

Skin protector with acrylic tape border

## Base Plate for 2-Piece system

### 1. Full hydrocolloid skin barrier for optimal skin protection


### 2. Combined hydrocolloid & acrylic tape border for extended weartime **NEW**

Discreet flange including belt loops for extra fixation when needed

### Excellent locking & easy coupling system

- Easy coupling with minimum pressure on the abdomen
- Extra reassurance: audible click will assure you that the pouch is securely positioned

Release paper with printed cutting-target

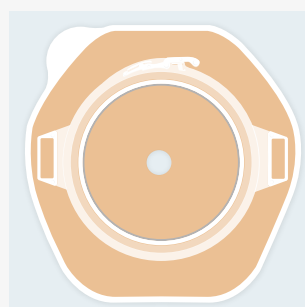


1. Full hydrocolloid




2. Combined hydrocolloid + acrylic tape border


Extended wear time & skin protection

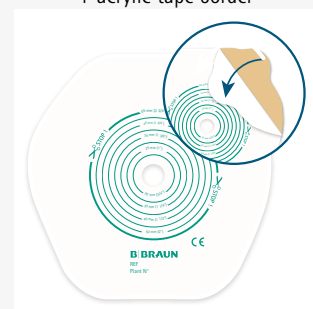


Release paper


# Proxima® 1-Piece






	Closed filter pouches (box of 30)	Diameter (mm)	References beige	References transparent
	Proxima	10-70	● 55010A	○ 55110A
	Proxima +	10-70	● 54010A	○ 54110A
<b>NEW</b>	Proxima +	25	● 54025A	-
	Proxima +	30	● 54030A	-
	Proxima +	35	● 54035A	-
	Proxima +	40	● 54040A	-
	Proxima +	45	● 54045A	-

	Drainable pouches (box of 30)	Diameter (mm)	References beige	References transparent
	Proxima*	10-70	● 55410A	○ 55510A
	Proxima Border*	10-70	● 56410A	○ 56510A
<b>NEW</b>	Proxima* +	10-70	● 54210A	○ 54310A


\*30 Soft clamp + 1 rigid


	Drainable filter pouches (box of 10)	Diameter (mm)	References beige	References transparent
	Proxima + + 1 rigid clamp	10-70	● 54410A	○ 54510A
<b>NEW</b>	Proxima + + 1 rigid clamp	25	● 54425A	-
	Proxima + + 1 rigid clamp	30	● 54430A	-
	Proxima + + 1 rigid clamp	35	● 54435A	-


	Urostomy pouches (box of 10)	Diameter (mm)	References transparent
<b>NEW</b>	Proxima Uro	12-55	○ 55540A


	Flat base plate (box of 10)	Diameter (mm)	References
	Proxima 2	 40/ 12-35	73040A
	Proxima 2	 50/ 12-45	73050A
	Proxima 2	 60/ 12-55	73060A
	Proxima 2	 80/ 12-75	73080A






# Proxima® 2-Piece

	Closed filter pouches (box of 30)	Diameter (mm)	References beige	References transparent
	Proxima 2	40	● 73140A	○ 73240A
	Proxima 2	50	● 73150A	○ 73250A
	Proxima 2	60	● 73160A	○ 73260A
	Proxima 2	80	● 73180A	○ 73280A
<b>NEW</b>	Proxima 2+	40	● 74140A	-
	Proxima 2+	50	● 74150A	-
	Proxima 2+	60	● 74160A	-
	Proxima 2+	80	● 74180A	-

	Drainable pouches (box of 30)	Diameter (mm)	References beige	References transparent
	Proxima 2 + 1 rigid clamp	40	● 73340A	○ 73440A
	Proxima 2 + 1 rigid clamp	50	● 73350A	○ 73450A
	Proxima 2 + 1 rigid clamp	60	● 73360A	○ 73460A
	Proxima 2 + 1 rigid clamp	80	● 73380A	○ 73480A

	Drainable filter pouches (box of 10)	Diameter (mm)	References beige	References transparent
	Proxima 2+ + 1 rigid clamp	40	● 74340A	○ 74440A
<b>NEW</b>	Proxima 2+ + 1 rigid clamp	50	● 74350A	○ 74450A
	Proxima 2+ + 1 rigid clamp	60	● 74360A	○ 74460A
	Proxima 2+ + 1 rigid clamp	80	● 74380A	○ 74480A

	Urostomy pouches (box of 10)	Diameter (mm)	References transparent
	Proxima 2 Uro	40	○ 73540A
	Proxima 2 Uro	50	○ 73550A
	Proxima 2 Uro	60	○ 73560A

	Flat base plate Border (box of 10)	Diameter (mm)	References
<b>NEW</b>	Proxima 2 +	 40/ 12-35	76040A
	Proxima 2 +	 50/ 12-45	76050A
	Proxima 2 +	 60/ 12-55	76060A
	Proxima 2 +	 80/ 12-75	76080A

This document, its contents, including institutional data, information, trademarks and logos mentioned herein are the exclusive property of B. Braun. Any representation and / or reproduction, total or partial, of this document and its contents without the express prior consent of B. Braun, is strictly prohibited and constitutes an infringement of the intellectual property rights of B. Braun.

Commercialized and manufactured by :  
 B. BRAUN MEDICAL SAS | 26 Rue Armengaud | 92210 Saint-Cloud - France  
 Tel. 01 41 10 53 00 | Fax 01 41 10 75 68 | www.bbraun.com  
 B. Braun Medical SAS with a share capital of 31 000 000 € | RCS Nanterre 562050856



For most countries a specific letter is added at the end of the radical reference.  
 All volumes are measured according to ISO 8670-2 standard.  
 This medical device is a regulated health care product and on the basis of this regulation is certified CE.  
 The content of this document, in particular the data, the information, the trademarks and the logos are B. BRAUN MEDICAL SAS's sole property and are protected under copyright ©2015, B. BRAUN MEDICAL, all rights reserved.  
 Consequently, all representation and/or reproduction, whether in part or in full, is forbidden and would be considered a violation of B. BRAUN MEDICAL SAS's copyright and other proprietary rights.  
 The products commercialized by B. BRAUN MEDICAL shall exclusively be used in accordance with the package inserts which have been updated and included in the boxes.

<https://www.bbraun.com/ostomy-care>